


Perfusor Space Troubleshooting Guide



Alarm Displayed	Action Required
Syringe invalid	<p>The syringe has been loaded incorrectly.</p> <ul style="list-style-type: none"> Remove the syringe. Insert the syringe with wings upright. Ensure the green clasp is situated to the right of the wings. Close the syringe clamp. The drive head will advance and automatically grip the syringe.
Pressure high	<ul style="list-style-type: none"> Press OK to acknowledge alarm. Check if tubing is kinked or damaged. Ensure tubing is not jammed on Space Perfusor door.
Syringe drive did not automatically close when pump is switched off	<p>The syringe holder was not closed when the pump was switched off.</p> <ul style="list-style-type: none"> Turn the pump on then close the syringe holder and switch the pump off by depressing on/off button for 3 seconds. The syringe drive will now close.
Pump not switching off	<p>Pump will not switch off with consumable still in pump.</p> <ul style="list-style-type: none"> Remove consumable and close clamp. Press power button for 3 seconds.
Battery empty	<ul style="list-style-type: none"> Press OK to acknowledge alarm. Plug into mains immediately or consider changing to another pump.
Battery Maintenance	<ul style="list-style-type: none"> Press OK and . The discharge process will start. Total duration of battery maintenance is approximately 12 hours.