



### DASH 4000 MONITOR: QUICK REFERENCE

#### **Admit & Activate Alarms:**

Press orange Silence Alarm/Admit button.

#### **Change Alarm limit:**

Use trim knob to highlight parameter, press to enter  
Highlight parameter limits, press to enter  
Highlight limit you want to change, press to enter  
Rotate trim knob to required limit  
Press to confirm.

#### **Remove Alarm Parameter Box**

Highlight MORE MENUS, press to enter  
Highlight MONITOR SET-UP, press to enter  
Highlight PARAMETERS ON/OFF, press to enter  
Rotate trim knob to move > to alarm, press to enter  
Rotate trim knob to move > to return, press to confirm

#### **Silence alarms:**

Press orange Silence Alarm key for 1 minute silence  
Press Silence Alarm key again for a 5 minute Alarm pause  
Press again to reactivate.

#### **Change ECG display LEAD**

Highlight ECG, press to enter  
Highlight DISPLAY LEAD, press to enter  
Select lead for ECG display, press to enter  
Highlight RELEARN, press to enter

#### **Change Alarm Volume**

Use trim knob to highlight alarm, press to enter  
Highlight ALARM VOL, press to enter  
Rotate trim knob to required level  
Press to confirm



#### **Start a BP Recording:**

Press grey NBP Go/Stop

To set **AUTOMATIC** recordings

Use trim knob to highlight NBP, press to enter

Highlight NBP AUTO, press to enter

Rotate trim knob to required setting.

Press to confirm

#### **ZERO** invasive pressures

Open line to air,

Press ZERO ALL grey key on front of monitor

When numbers on screen say zero, close

line to air

#### **Accessing Patient Data:**

Highlight MORE MENUS, Press to enter

Highlight PATIENT DATA, Press to enter

Highlight requirement information box,  
e.g. VITAL SIGNS

Press to enter

#### **DISCHARGE** (This clears all previous data)

Highlight MORE MENUS, press to enter

Highlight ADMIT MENU, press to enter

Highlight DISCHARGE PATIENT, press to enter

Rotate trim knob so cursor points to

DISCHARGE

Press to confirm

On Call Electronics Technician Dial

Pager no 93-4203

Mobile 025 2919461

Cass Medical phone 0800 122277