


Getting Started Perfusor® Space Syringe Pump



**This Quick Guide
does not replace the
Instructions for Use!**


1 Turn the pump on by pressing 




2 After the pump has finished its self checks and the drive arm has extended, open the syringe holder and turn fully to the right, then pull the door open.





3 Insert the syringe, ensuring the flange on the syringe is on the left of the green clip and in a vertical position. Now close the syringe holder.




4 Confirm the syringe make and size by pressing 

- Once in the main menu, press  to open rate setting.
- Select the correct rate with .
- Press  to commence infusion.

5 To Change the syringe

- Press  to stop the infusion.
- Open syringe holder and close again.
- Answer "YES" with  to "Perform syringe change?"
- Open syringe holder and pump door; remove syringe.
- Insert new syringe or turn pump off.

6 Turn the pump off by closing the syringe holder and door, then press and hold 

Attempting to remove the syringe without following the correct procedure may result in pump damage.